



Face Shield

DESCRIPTION

Face shields are a personal protective equipment (PPE) device used for infection control. A face shield is required when face protection is needed. It is used to protect against potential contamination such as

splashes, spray, splatter or droplets of blood or other potentially infectious materials (OPIM). It will provide protection of the eyes, nose and mouth.

FEATURES AND BENEFITS

- ✓ Splash resistant
- ✓ Anti fog
- ✓ Light-weight
- ✓ Elastic strap to suit different head sizes
- ✓ Optically clear and no dizziness
- ✓ Vented foam headband relieves pressure on forehead

PRODUCT DATA

- 32cm (W) x 22cm(L)
- 100% PET film
- 100% Transparent
- PU foam liner
- Elastic headband

INSTRUCTIONS FOR USE

Donning a face shield:

- Remove protective film on lens before use.
- Place the foam against your forehead and stretch the elastic band over your head.
- Once the face shield is on, check to ensure it fully covers the front and sides of the face and no areas are left uncovered.

Removing a face shield:

- To remove, pull the elastic strap forward and over your head. Be careful not to touch the front of the face shield.
- Dispose of it immediately in a hazardous waste container.

WARNING:

- Do not use as industrial PPE.
- This shield is not impact resistant.
- Do not use as protection against welding or grinding processes.

Personal Protective Equipment

RE-ORDER	DESCRIPTION	SIZE (CM)	STERILE / NON-STERILE	PIECES PER BAG	BAGS PER CARTON	TOTAL PIECES PER CARTON
54-104NS	Face Shield	32cm x 22cm	Non-Sterile	1pc	200 Bags	200pcs