



N95 Surgical Respirator Level 3 (54-407C)



DESCRIPTION

N95 respirators are a respiratory protective device intended to protect the wearer from airborne particles. When fitted correctly, the respirator forms a seal around the mouth and

nose and can filter a minimum of 95% of airborne particulates. This N95 respirator also provides AAMI level 3 (160 mmHg) fluid resistance protection.


PRODUCT DATA

- 15cm (L) x 12.5cm (W)
- 3 layer construction
- Latex free
- Non-sterile
- ARTG 335090
- NIOSH approval number TC-84A-8109


FEATURES AND BENEFITS

- ✓ CFR 84.180 (NIOSH N95) compliant
- ✓ ASTM F2100-2019 level 3 (160 mmHg) compliant
- ✓ Particulate filtration efficiency (PFE) >= 95%
- ✓ 2 head loop straps
- ✓ Cupped shape
- ✓ Single use
- ✓ Malleable nose strip with moisture trapping foam to reduce fogging


FITTING INSTRUCTIONS

1 


1. Cup the respirator in your hand with the nose piece at your fingertips allowing the headbands to hang freely below your hand.

2 


2. Cup the respirator firmly against your face with the nose piece on the bridge of your nose, Pull the bottom strap over your head positioning it around the neck below ears.

3 

3. Pull the top strap over your head resting it high at the top back of your head.

4 

4. Press soft nose piece to conform comfortably around the nose.

5 

5. Check for leakage (facial fit) as follows:

- Place both hands over the respirator and exhale sharply
- Positive pressure should be inside the respirator
- If you feel air escaping around the edges, re-adjust the respirator by tightening the nose-piece
- Repeat the procedure until satisfactory face-fit has been achieved.

Personal Protection Equipment

RE-ORDER	DESCRIPTION	SIZE (CM)	STERILE / NON-STERILE	PIECES PER INNER BOX	INNERS PER CARTON	TOTAL PIECES PER CARTON
54-407C	N95 Surgical Respirator, Level 3, Cupped Style, Head Loops	15cm x 12.5cm	Non-Sterile	20pcs	20 Boxes	400pcs