



Personal Protection Equipment

P2 Respirator (54-409D)



DESCRIPTION

P2 respirators are a respiratory protective device intended to protect the wearer from airborne particles. When fitted

correctly, the respirator forms a seal around the mouth and nose and can filter a minimum of 94% of airborne particulates.

PRODUCT DATA

- 20cm (L) x 16cm (W)
- 4 layer construction
- Latex free
- Non-sterile
- ARTG 350735

FEATURES AND BENEFITS

- ✓ AS/NZS 1716:2012 compliant
- ✓ Particulate filtration efficiency (PFE) >= 94%
- ✓ 2 adjustable head loop straps for better securement
- ✓ Single use
- ✓ Individually wrapped
- ✓ Malleable nose strip with moisture trapping foam for comfort

FITTING INSTRUCTIONS

1

1. Cup the respirator in your hand with the nose piece at your fingertips allowing the headbands to hang freely below your hand.

2

2. Cup the respirator firmly against your face with the nose piece on the bridge of your nose, Pull the bottom strap over your head positioning it around the neck below ears.

3

3. Pull the top strap over your head resting it high at the top back of your head.

4

4. Press soft nose piece to conform comfortably around the nose.

5

5. Check for leakage (facial fit) as follows:

- Place both hands over the respirator and exhale sharply
- Positive pressure should be inside the respirator
- If you feel air escaping around the edges, re-adjust the respirator by tightening the nose-piece
- Repeat the procedure until satisfactory face-fit has been achieved.

Personal Protection Equipment

RE-ORDER	DESCRIPTION	SIZE (CM)	STERILE / NON-STERILE	PIECES PER INNER BOX	INNERS PER CARTON	TOTAL PIECES PER CARTON
54-409D	P2 Respirator, Headloops	20cm x 16cm	Non-Sterile	90pcs	12 Boxes	1080pcs