

# Protective Gowns



## Isolation Gown (Thumb Secured Lite-back)

RE-ORDER  
29-291  
29-293

### DONNING GOWN



1  
Perform hand hygiene. Make sure gown is not damaged or contaminated before donning.



2  
Holding the upper part of the gown, flick & drop gown in front ready for donning.



3  
Slip one arm into sleeve and place thumb into thumb loop. Repeat for the other arm & thumb.



4  
Fasten the fastener at back of neck, making sure they are generously overlapped.



5  
Grabbing both ends of the tie, wrapped it around the waist and securely tie at back. Then, put on mask, eyewear & gloves.


### DOFFING GOWN



1  
Remove & discard gloves. Perform hand hygiene. Then unfasten back neck fastener & waist tie.



2  
Gown front & sleeves are contaminated. So, pull gown away from body, touching only the inside of gown.

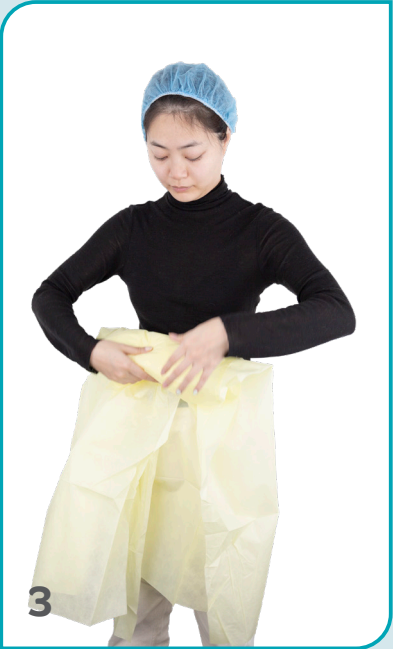

3  
Turn gown inside out by folding and rolling gown into bundle without touching the external part of the gown.



4  
Properly discard of contaminated gown into clinical waste. Perform hand hygiene. Then remove eyewear & mask.



5  
Perform hand hygiene again after removing & discarding all PPE into clinical waste.

### CAUTIONS / PRECAUTIONS

To reduce the risk of transmission of infectious agents, PPE must be use/worn appropriately.