

Notched Tourniquet

Tighten infection control.

Our button tourniquet is a universal, cost effective solution to venous access. As a single use item, it helps to reduce the risk of cross-contamination and improve infection control and the patient hospital experience.

RE-ORDER
25-112NS

100 pieces / box
20 boxes / carton

DESIGN

With growing evidence that tourniquets are a source of MRO transmission and cross-infection, our single use tourniquet has been designed to complement meticulous sterile technique.

- ✓ Universal solution for adults and paediatrics
- ✓ Easy to use
- ✓ Quick release
- ✓ Reduces cross contamination
- ✓ Improves patient comfort



KEY FEATURE


Simple to use.

Designed around a simple three step process, our single-use tourniquet ensures efficient application and a quick one-hand release.


Note: If a tourniquet has been applied for longer than a minute, it should be released and re-applied after 2 minutes.



1 Wrap tourniquet around the arm, 7.5-10cm above puncture site and thread one end through the slit.

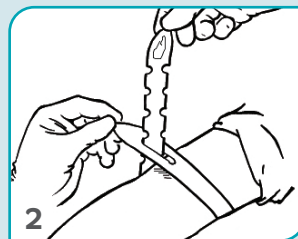

2 Keep pulling the tourniquet until the desired tension has been achieved. The tourniquet will then lock in position.

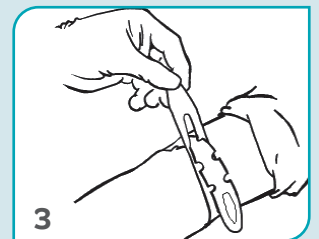

3 To release the tourniquet, pull the short end of the tourniquet and it will release immediately.

CAUTIONS / PRECAUTIONS

Do not apply high pressures that may impair arterial blood flow to the extremity.

Release tourniquet as soon as blood starts to flow into the first tube.

Too long and too high stasis may contribute to haemolysis.