


MULTIGATE
Vienna™





Addressing clinician concerns about measuring and selecting the right stocking for patients, we created the Vienna™ system which simplifies the storage and selection of anti-embolism stockings, ensuring every stocking is measured and fitted with ease.



VTE is a major health problem in Australia, costing the Australian healthcare system an estimated \$1.7 billion every year.



Multigate Vienna™
Anti-Embolism
Stockings are an
effective method
of DVT prevention.

60%

less
DVT
incidence



The Vienna™ system makes it easy to fit patients with the correct stockings with the correct stockings.

A simple colour coded system that requires only two (for knee high stockings) or three (for thigh high stockings) measurements.



MULTIGATE Vienna™ Anti-Embolism Stockings

Perfect fit every time — the Vienna system makes it easy to fit patients with the correct stockings. With less points of measurement and a simple colour coded system, it's easy to select the right product first time, every time.

Knee

To determine size for knee high stockings:

- Measure ankle circumference to determine the size.
- Measure calf circumference to ensure the limb fits within the parameters of the determined size.

SIZE	CODE	ANKLE CIRCUMFERENCE (cm)	CALF CIRCUMFERENCE (cm)	LEG LENGTH (cm)
XS	68-300	19-22	25.5-34	30-39
S	68-301	22-25	33-40	30-39
M	68-311	25-28	38-45	37-44
L	68-321	28-31	43-50	37-44
XL	68-331	31-34	44-53	37-44
XXL	68-341	34-37	49-56	40-45
XXXL	68-351	37-40	53-64	40-45

To apply knee high stockings:

- Insert hand into stocking down to heel pocket.
- Grasp heel pocket and turn stocking inside out to heel area.
- Carfully pull stocking over foot and heel.
- Pull stocking up to about 3-5 cm below bottom of knee cap. Smooth excess material out.

CAUTION:

- Do not turn down top of stocking
- Do not cover any part of the knee
- Ensure the use of correct size stocking

Thigh

To determine size for thigh high stockings:

- Measure ankle circumference to determine the size.
- Measure calf circumference to ensure the limb fits within the parameters of the determined size.
- Measure thigh circumference to ensure the limb fits within the parameters of the determined size.

SIZE	CODE	ANKLE CIRCUMFERENCE (cm)	CALF CIRCUMFERENCE (cm)	THIGH CIRCUMFERENCE (cm)	LEG LENGTH (cm)
XS	68-400	19-22	25.5-34	44-54	42-70
S	68-401	22-25	33-40	50-62	42-70
M	68-411	25-28	38-45	54-68	70-80
L	68-421	28-31	43-50	62-74	70-80
XL	68-431	31-34	44-53	68-80	70-80
XXL	68-441	34-37	49-56	74-84	75-83
XXXL	68-451	37-40	53-64	80-92	75-83

To apply thigh high stockings:

- Insert hand into stocking down to heel pocket.
- Grasp heel pocket and turn stocking inside out to heel area.
- Carfully pull stocking over foot and heel.
- Pull the remaining stocking up to the top of the thigh. The thigh band sits 1cm below the gluteal furrow. Smooth out the excess material and ensure there are no creases.

CAUTION:

- Do not turn down top of stocking
- Ensure the use of correct size stocking

Contraindications

Stockings are not recommended for patients with the following conditions:

- X Any leg condition in which stockings would contraindicate, such as:
 - a. dermatitis
 - b. vein ligation (immediate postoperative)
 - c. gangrene
 - d. recent skin graft
- X Severe arteriosclerosis or any other ischaemic vascular disease
- X Massive oedema of legs or pulmonary oedema from congestive heart failure
- X Extreme elasticity of leg

A measurement ruler is included with every pair of stockings, so it is always at hand when needed.

Simple observation
Patient observation is made
easier by an inspection hole
at the ball of the foot.



With its slim packaging and optimised sizing range, the Vienna™ solution ensures efficient storage while still catering for bariatric needs and the full array of size requirements.





The entire Vienna™ range is latex free, making it suitable for patients allergic to latex and designed to wick perspiration away from the skin keeping the patient cool and comfortable.



MULTIGATE
Vienna™



Perfect fit. Every time.

